

Cumulative change log for DentalEye 3

DentalEye 3.4.5

21 resolved issue

ID	Summary	Type	Priority	Component
#1308	Clearer indication for annotation mode	Improvement	Medium	Main program
#1822	Installer no longer mixes up DMII with DM3	Bug report	High	Installer
#1824	Error message no longer shown when searching and opening patients in Enterprise with PACS selected for Q/R.	Bug report	Critical	Main program
#1826	DB connection no longer fails if Databasetype reg key is missing	Bug report	Critical	Main program
#1832	\x path no longer stored if file opened from wrong folder	Bug report	Medium	Main application
#1833	Improved performance for large DB queries with cloud DB	Bug report	Medium	Main program
#1834	Web viewer can now show annotations	Improvement	Critical	Web app frontend
#1835	Added basic export function in Web viewer.	Improvement	High	Web app frontend
#1836	Series no longer risk losing template association when retrieved from PACS.	Bug report	Critical	Main program
#1837	Import preview no longer too light for color and 8 bit gray	Bug report	High	Main program
#1838	Click outside textbox when editing annotation text no longer causes crash	Bug report	Critical	Main program
#1839	Selected background color now shows in review mode too	Bug report	Medium	Main report
#1841	User names with period or space can now successfully be used with cloud storage	Bug report	Critical	Cloud backend
#1842	Export now submits to selected GUI theme regardless of image selection	Bug report	High	Main application
#1849	Patient folder p.json update no longer autotriggers and is much faster	Improvement	Critical	Main program
#1851	Zoomed images now prints properly again	Bug report	Critical	Main program
#1853	Enabled install from MS Store to improve defender smart app control rumor	Improvement	Critical	Installer
#1854	Fixed issue with error messages when capturing images with templates having many frames	Bug report	Critical	Main program
#1855	Zoom applied to wrong images after capture session that includes deletes.	Bug report	Critical	Main program
#1861	Image LUT no longer applied to info label on export to file	Bug report	Critical	Main program
#1863	Annotations are now stored to and retrieved properly from PACS again	Bug report	Critical	Main program

DentalEye 3.4.4

44 resolved issues

ID	Summary	Type	Priority	Component
277	Default fore- and back color of annotations can be set using a registry key.	Improvement	Low	Main program
293	Fixed weird behavior when importing mix of images and non-image objects in Pro.	Bug report	Low	Main program
354	Advanced enhancement filters now work ok with DICOM files with monochrome1 photometric interpretation.	Bug report	Low	Main program
802	Manual Import plugin no longer stops working after one failed import	Bug report	Low	Manual import plugin
879	Anatomic info is no longer cleared from image when dropped on other image in template frame with anatomic info NONE.	Bug report	Medium	Main program
930	Using empty string as imagecard name is no longer accepted	Bug report	Low	Main program
1265	12 bit grayscale data now stored ok in lossy J2K DICOM files.	Bug report	Medium	Main program
1266	Non-functional "perceptual gamma" settings removed.	Bug report	Low	Main program
1352	Export "all images on patient" is now available when it should, not just for the first patient opened.	Bug report	Low	Main program

1355	Added option to ignore incoming blank images non-intrusively	Improvement	Medium	Main program
1358	Importing many high resolution photos into the same series no longer fails to save and show as X-images.	Improvement	Critical	Main program
1375	Reset in B/G tool now resets gamma properly	Bug report	High	Main program
1382	Windows 11 graphics issues in some controls fixed	Improvement	Low	Main program
1472	Annotations can now be deleted with the delete key regardless of approved state.	Bug report	Low	Main program
1553	Imported signed 12/16 bit grayscale images now display ok	Improvement	Critical	Main program
1609	Export plugin will now warn about missing destination folder	Improvement	Low	Export plugin
1614	Auto converting from non-DCM to DCM in Enterprise now works again.	Bug report	Critical	Main program
1627	Backslash test on startup removed	Improvement	Medium	Main program
1667	Added Dentsply Sirona plugin supporting IOSS drivers	Improvement	Critical	Dentsply Sirona plugin
1682	'DRAFT' on splash no longer shows for bad release version installs	Bug report	Low	Main program
1691	Added Handy plugin	Improvement	Medium	Handy Plugin
1697	Some plugin installers no longer trigger error message on disks larger than 4TB.	Bug report	Medium	AutoImport plugin C-Takt plugin Export plugin Manual import plugin Sirona plugin TWAIN plugin
1698	Extended integration with Almasoft TLS now works again	Bug report	Critical	PMS link
1707	Added cloud storage option for desktop client	Improvement	Critical	Main program
1708	A web viewer app can be used with the cloud storage option	Improvement	Critical	Web app frontend
1717	Dialogs now shows on the same screen as the main application window in Windows 11.	Bug report	Medium	Main program
1728	Support for non-linear VOI LUT on import and export for 12/16 bit gray.	Improvement	Medium	Main program
1732	Panorama zoom window function removed	Improvement	High	Main program
1735	Annotations now included in mirror operations	Bug report	Medium	Main program
1736	Sirona plugin no longer prevents correct patient info in device display.	Bug report	Medium	Sirona plugin
1738	Many captures during same session no longer leads to X-image due to memory issues.	Bug report	Medium	Main program
1751	Images left behind by previous session no longer imports automatically on startup in VistaEasy plugin.	Improvement	Critical	Dürr plugin
1752	Image import limited to current job in VistaEasy plugin	Improvement	Critical	Dürr plugin
1755	TWAIN plugin no longer attempts to close session on patient selection	Improvement	Medium	TWAIN plugin
1756	Minimized risk of TWAIN plugin crash with bad drivers	Improvement	Medium	TWAIN plugin
1757	Manual import plugin dialog now looks and works better	Improvement	Medium	Manual import plugin
1761	Dentail plugin supports new API features and autosend on series approval	Improvement	High	Dentail plugin
1764	Left arrow key and left arrow screen button now moves selection in expected direction again.	Bug report	Critical	Main program
1769	Carpo dental plugin now applies corrections properly	Improvement	High	Carpo dental plugin
1778	Shows info about proportions in status bar during crop	Improvement	Medium	Main program
1780	References to Windows 10 support removed from IFU and II	Improvement	Critical	Documentation
1782	Patient names are now updated from MWL query results	Bug report	Critical	Tool
1783	Logging extended when selecting MWL items	Improvement	High	Tool
1809	New option to turn off auto optimization in Sirona plugin. It also fronts the capture dialog better with Sidexis 4.	Improvement	High	Sirona plugin

DentalEye 3.4.3

1 resolved issue

ID	Summary	Type	Priority	Component
1684	Un-selection of teeth now gets properly stored to DICOM dataset with Primary anatomic structure sequence setting.	Bug report	critical	Main program

DentalEye 3.4.2

24 resolved issues

ID	Summary	Type	Priority	Component
1674	Modality worklist query window can now be resized	Improvement	high	Tool
1666	Referrers' computer name and clinic is now shown in control book.	Improvement	high	Main program
1662	Upgrade to new GUI components in main projects	Improvement	high	Main program
1659	Customization dialog is now translated into danish and Swedish	Bug report	high	Main program
1655	Review mode image order is now always chronological.	Improvement	medium	Main program
1654	L/R arrow keys can now be used to move selection to next or previous image/images. Works for multi-selection scenarios and compares too.	Improvement	medium	Main program
1652	CmdLink no longer removes additional first names passed using /LF command line option.	Bug report	medium	PMS Link
1651	Clinic is now shown in properties pane.	Improvement	medium	Main program
1650	Installer issue when adding certain plugins resolved	Bug report	high	Installer
1649	Support added for newer Ez3D-i viewer in Imaging link plugin.	Improvement	medium	Plugin: Imaging link
1648	Meta header 0002,0000 is now included or not where expected.	Bug report	high	Unspecified
1647	Installer exe is now digitally signed.	Improvement	high	Installer
1644	Added Vatech Ez3D support to Imaging link plugin	Improvement	medium	Plugin: Imaging link
1643	Added workaround for import of Panoramic and CBCT-series with mA value in Study UID (!) from old Vatech devices.	Improvement	medium	Plugin: Vatech
1636	Add option to strip hyphen from PID in CmdLink	Improvement	critical	PMS Link
1632	Added support for Synedra viewer to Imaging link plugin	Improvement	medium	Plugin: Imaging link
1629	Added support for 0 & 1.5 sensor sizes in Carpo dental plugin	Improvement	medium	Plugin: Carpo dental
1615	Accession number prefix option added.	Improvement	medium	Main program
1611	Hanging protocols improved for imported DCM files with primary anatomic structure toothing.	Improvement	high	Main program
1486*	Instructions for use updated to comply with MDR	Improvement	critical	Documentation
1485*	Labeling updated to comply with MDR	Improvement	critical	Main program
1405	E-mail with 64 bit e-mail clients now works	Improvement	medium	Main program
1376	Gray level no longer wrong directly after 16 bit image capture in Enterprise version.	Bug report	medium	Main program
429	Updated tooth chart and improved ability to select anatomic info in properties pane.	Improvement	low	Main program

* Implemented already in internal-only release 3.4.1.

DentalEye 3.3.6

12 resolved issues

ID	Summary	Type	Priority	Component
1585	Progeny plugin installs and runs properly on 64 bit OS.	Bug report	high	Plugin: Progeny
1572	Windows 11 support	Improvement	critical	System
1571	Carpo dental plugin	Improvement	low	Plugin: Carpo dental
1532	Sopix2 image orientation is now correct	Bug report	critical	Plugin: Acteon
1480	Dürr VistaEasy plugin now works properly with latest	Improvement	critical	Plugin:

	VistaSoft release.			Dürr VistaEasy
1457	Acteon U-Sense plugin (IngxCliD SDK)	Improvement	critical	Plugin: Acteon Intraoral, Owandy Intraoral
1439	PSPix2 now able to produce 16 bit images properly	Bug report	high	Plugin: Acteon
1381*	Sysinfo now detects Windows 11 properly	Bug report	medium	Tool
1378	Dentail service support (Dentail plugin)	Improvement	high	Plugin: Dentail
1365*	Correct text about moving images in Instructions for use.	Bug report	medium	Documentation
1351*	Update driver link in Schick plugin appendix of Installation.	Improvement	medium	Documentation
1	Migrate to Windows installer for DE 3.3	Improvement	medium	Installer

* Implemented already in internal-only release 3.4.1.

DentalEye 3.3.5

2 resolved issues. No functional changes.

ID	Summary	Type	Priority	Component
1368	New office address in IFUs and II	Improvement	critical	Documentation
1367	New office address in about/splash	Improvement	critical	Main program

DentalEye 3.3.4

5 resolved issues

ID	Summary	Type	Priority	Component
1338	DESysInfo fails to properly detect Windows 10 20H2	Improvement	medium	Tool: Sysinfo
1336	Dropped Windows 7 support for Camera WIA plugin	Improvement	low	Plugin: CameraWIA
1334	Improve Planmeca IO capture speed	Improvement	medium	Plugin: Planmeca
1333	Planmeca plugin setup falls behind and has no manifest on exe	Bug report	high	Plugin: Planmeca
168	Planmeca plugin: Sensor serial number not updated when new sensor is connected	Improvement	medium	Plugin: Planmeca

DentalEye 3.3.3

20 resolved issues

ID	Summary	Type	Priority	Component
1327	Scripted test findings	Bug report	high	-
1326	Non scripted findings	Bug report	medium	-
1319	Improved error handling for SaveWindowPos	Bug report	medium	Main program
1289	Prevent move of menu	Improvement	medium	Main program
1288	DPI not properly retrieved on all DCM file imports in PRO.	Bug report	high	Main program
1286	DESysinfo should support use via script.	Improvement	low	Tool
1285	CS 6100 capture fails if activated using cable button.	Bug report	medium	Plugin: Carestream
1275	ExecuteIPCMessgae2 error: 3711	Bug report	high	Main program
1272	Study date 1899 with PMS link on new unapproved series.	Bug report	critical	Main program
1271	DEViewer not properly working in 3.3.2	Bug report	high	PMS Link
1267	Error: modFunction.ApproveStation on workstation approval	Bug report	high	Main program
1264	Installation instructions for 3.3.3 update	Improvement	medium	Documentation
1250	Schick plugin needs reinstall after W10 feature update	Bug report	medium	Plugin: Schick
1247	Error: MDIMain.tmrDelayedOperation error: 91, 65	Bug report	high	Main program
1225	Automatic move to next frame not always working	Bug report	medium	Main program
1041	AutoImport should have no of images limit per import.	Improvement	low	Plugin: Autoimport
987	Camera WIA handling of bad devices	Improvement	critical	Plugin: Camera WIA
894	SchickLink appears in registry for no reason	Bug report	low	PMS Link
495	Prevent users without rights to open application	Bug report	low	Main program

319	Planmeca plugin process hung	Bug report	low	Plugin: Planmeca
-----	------------------------------	------------	-----	------------------

DentalEye 3.3.2

22 resolved issues

ID	Summary	Type	Priority	Component
1263	Non scripted test findings for 3.3.2	Bug report	high	Main application
1255	Remote plugins	Improvement	medium	Plugin: Remote
1254	Vatech EZSensor plugin	Improvement	high	Plugin: Vatech IO
1251	Centralized Workstation approval for Enterprise systems	Improvement	high	Main program
1249	Event log not shown properly if containing 3.1 rows.	Bug report	critical	Tool
1231	Allow licenses beyond 2021-04-30	Improvement	critical	Main program
1214	Update AutoImport in 3.3.2	Improvement	high	Plugin: AutoImport
1210	Developer dialog shows only one row with > 36 F-keys	Bug report	high	Main program
1209	cmdLink should be able to handle background selection and closing of patient just like DELink	Improvement	low	PMS Link
1208	Closing patient with DELink causes unnecessary timeout	Bug report	high	PMS link
1207	Imagelevel MDX3 plugin not properly detected by 3.3.1 installer	Bug report	high	Installer
1204	Size of Overflow pane not remembered properly when launched from PMS link	Bug report	high	Main program
1203	Error message "frmMain.ControlSizing error: 5" Invalid procedure call or argument	Bug report	medium	Main program
1198	Slow when opening patient after using 3.3.1 for a while.	Bug report	critical	Main program
1194	Problems with Morita and PMS link in 3.3.1	Bug report	medium	Plugin: Morita
1179	Unable to select "All images on patient" in Export dialog	Bug report	medium	Main program
1172	Wrong rotation with Kavo Scan eXam in KavoKerr plugin	Bug report	medium	Plugin: Kavo Kerr
1130	Template changes not possible to save after click on other template, no in save dlg and restart on "New Template"	Bug report	insignificant	Main program
1079	DCM file support in AutoImport	Improvement	medium	Plugin: AutoImport
961	Header sorting in Monitor calibration dialog	Improvement	medium	Main program
372	Planmeca plugin IO capture is too slow	Improvement	medium	Plugin: Planmeca
176	Add mirror as a property of the template frames.	Improvement	medium	Main program

DentalEye 3.3.1

84 resolved issues

ID	Summary	Type	Priority	Component
1161	Non scripted test findings for 3.3.1	Bug report	high	Main program
1159	Code review findings from 2019-11-21	Improvement	medium	Unspecified
1157	HaS1168X plugin (Hamamatsu 2) issues with W10 > 1903	Bug report	medium	Plugin: Hamamatsu
1138	Ability to load entire CT series at will even if limited	Improvement	medium	Main program
1137	Some DICOM files from Sectra cause error message after import.	Bug report	medium	Unspecified
1134	Gamma-korr does not show properly if presentation view from imagelist	Bug report	medium	Main program
1129	Update Instructions for use for 3.3.1 release	Improvement	critical	Documentation
1128	Update Installation instruction for 3.3.1 release	Improvement	critical	Documentation
1119	Preamble security issue with DICOM-files	Improvement	medium	Main program
1118	CS Adapt support for RVG in Carestream plugin	Improvement	medium	Plugin: Kodak
1112	Error if opening Preferences when more than 100 users are active	Bug report	medium	Main program
1102	Bonoprox integration	Improvement	low	Unspecified
1096	Aero snapped main window will not return to snapped position	Bug report	low	Main program
1095	Edit image info from Image list fails to update other instance on Series or Imagecard	Bug report	high	Main program
1086	Update IFU regarding standards references	Improvement	medium	Documentation
1085	Implement UDI (Universal Device Identifier)	Improvement	high	Main program
1080	Image selection indicator not clear enough	Improvement	medium	Main program
1052	Fail to drag image from image list to series when series is shown with left and right margins.	Bug report	medium	Unspecified
1025	Remove Toolbar background image and main window background color/image settings.	Improvement	medium	Main program
1024	Remove ability to move task pane to right side,	Improvement	medium	Main program
1023	Vatech plugin	Improvement	critical	Unspecified
1001	Mousewheel zoom does not center on pointer position if possible	Bug report	medium	Main program
994	Mscmct2.dll vulnerability	Improvement	medium	Installer
992	Enhanced CBCT issues	Bug report	medium	Main program
989	Possible bug with mirror + rotate before approve	Bug report	low	Main program
973	DESysInfo need update to properly detect new windows 10 versions	Improvement	medium	Tool
971	Prompt for New or Remount defaults	Improvement	medium	Main program
968	Exposure parameters from Cranex Novus E not received in DE	Improvement	high	Plugin: Soredex
962	PSPix (old Optime OEM version) is rotating images differently than in 3.1	Improvement	medium	Plugin: Soredex
957	Schick USBCam 4 buttons not working	Improvement	medium	Plugin: Video
956	Video control fails to find window	Bug report	medium	Plugin: Video
955	C-takt plugin fails to preserve approved brightness and contrast	Bug report	high	Main program
952	Missing series and/or study object after approve	Bug report	high	Main program
950	Gendex plugin fails to import captured image with both sensor and camera connected.	Bug report	high	Plugin: Gendex
935	Apostrophe escaping missing in function cUser.DeleteDev	Bug report	high	Main program
933	Manual entry of license does not work	Bug report	high	Main program
931	Extra tabs missing in settings dialog of Soredex Digora Optime	Improvement	medium	Plugin: Soredex
924	Resize IC leads to missing selection in Export	Bug report	low	Main program
922	Undo all on a rotated image on IC fails to trigger save	Bug report	low	Main program
919	Monitor calibration on startup gets behind splash screen	Bug report	high	Main program
915	Image capture must be disabled in Presentation overview	Bug report	critical	Main program
910	New Progeny/Midmark plugin	Improvement	medium	Unspecified
908	CD burning folder confusion	Improvement	low	Main program
907	Exported auto file name should conform to custom PID label content	Improvement	medium	Main program
906	PID label disturbing when analyzing image in review mode	Design error	critical	Main program
902	Delete more than one image from (green) imagecard fails.	Bug report	high	Main program
889	Customizable PID label	Improvement	medium	Main program

888	“Do you want to remove patient from database” is confusing to the user and should be disabled.	Design error	high	Main program
885	Schick plugin - new enhancement option in new SDK	Improvement	medium	Plugin: Schick
884	Imaging link plugin add support for BlueSkyBio	Improvement	medium	Plugin: Imaging link
880	New SDK from "KaVo Kerr" for KaVo, Soredex and Instrumentarium devices	Improvement	high	Unspecified
878	Remove "Change approval" option from context menu	Improvement	medium	Main program
875	Add function to document proper installation/calibration of X-ray chain	Improvement	critical	Main program
873	Silent start-up of plugins when drivers and hardware not found	Improvement	medium	Unspecified
872	Add wrapper test case for generic test cases	Task	high	Unspecified
871	Plugin error logging	Improvement	medium	Unspecified
867	Date append in Series and Study name	Bug report	high	Main program
866	Image count in series list missing	Bug report	medium	Main program
864	Installer must update or remove plugins in 3.3.1. NOT leave 3.2.1!	Improvement	critical	Installer
863	Unattended Install with parameter presets	Improvement	high	Installer
859	SysInfo detects some files as changed due to daylight saving	Bug report	critical	Tool
850	MWL confirmation prompt missing in English language file	Bug report	high	Main program
848	Windows 10 support	Improvement	critical	Unspecified
847	Imaging link plugin for 3.2	Improvement	medium	Plugin: Imaging link
844	SysInfo license info missing	Bug report	low	Tool
841	Clean language files from unused strings and fix errors	Improvement	medium	Main program
832	Acteon plugin with support for Sopro Sopic 2 sensor and PSPix2 scanner.	Improvement	critical	Unspecified
830	SIDIIN locating in Sirona plugin	Improvement	high	Plugin: Sirona
829	New Opus integration improvement	Improvement	high	PMS Link
792	Missing functionality for CT image stacks.	Bug report	medium	Main program
760	Add a new annotation tool – Mach Effect Correction	Improvement	medium	Main program
758	MVRQ0062: Allow the user to specify a file name for single-file exports.	Improvement	medium	Main program
625	Remove tooth number tooltips from tooth chart.	Design error	low	Main program
613	Imaging link plugin changes for hosts 3.1.1385 and newer	Improvement	medium	Plugin: Imaging link
589	MVRQ0048 SimPlant support in Imaging Link	Improvement	medium	Plugin: Imaging link
580	MVRQ0047: Add option to prevent multiple thin client instances of MiPACS	Improvement	medium	Main program
368	Paste function in 3.1.1258 does not work as in 3.1.1025	Bug report	medium	Main program
360	DE32Thumbs to replace DEThumbs	Improvement	high	DEXTThumbs
236	Vatech plugin	Improvement	medium	Unspecified
169	Straighten up images, rotate in small steps	Improvement	high	Main program
164	Delete images from Clipboard	Bug report	low	Main program
154	Workstation name and Client AE in terminal server mode	Improvement	medium	Main program
117	Cannot import PDF files	Bug report	medium	Main program
2	New look and feel for DE 3	Improvement	critical	Main program

DentalEye 3.2.3

5 resolved issues

ID	Summary	Type	Priority	Component
1022	CS7600 scanner support in Carestream plugin	Improvement	high	Plugin: Kodak
923	Carestream 6200 sensor support	Bug report	critical	Plugin: Kodak
883	Carestream plugin fails with " (double quote) in patient name	Bug report	high	Plugin: Kodak
858	Carestream plugin should support new SDK versions	Improvement	medium	Plugin: Kodak
852	Carestream plugin 3D capture import issue	Improvement	medium	Plugin: Kodak

DentalEye 3.2.2

This release was cancelled with most of the issues planned for 3.3.1

DentalEye 3.2.1

97 resolved issues

ID	Summary	Type	Priority	Component
846	Danish language support	Improvement	critical	Unspecified
831	Hamamatsu S1168X/Dentalmind Digital X-ray 3 plugin	Improvement	critical	Unspecified
824	Code review finding from 2015-05-08	Improvement	medium	Main program
823	Code review findings from 2015-01-20	Improvement	low	Unspecified
822	Code review: Hard coded language strings.	Improvement	medium	Unspecified
821	Code review: Inverted logic in if-statements	Improvement	low	Main program
820	Code review: Unused code commented out makes it harder to follow the logical flow.	Improvement	low	Main program
818	Edit number in Enterprise version	Bug report	critical	Main program
813	Date of birth in printouts	Improvement	medium	Main program
812	Bug that can cause error 91 on save of new series	Bug report	critical	Main program
811	Miniviewer can't be included with 3.2.1	Task	critical	Main program
810	Bug with MWL Restriction of additional x-rays [IOXECM] 2	Bug report	medium	Main program
809	Bug with MWL Restriction of additional x-rays [IOXECM] 1	Bug report	medium	Main program
808	Missing error message about file access on approval	Bug report	medium	Main program
807	User instructions needed in error messages	Improvement	critical	Main program
805	Regulatory labelling in About-box	Improvement	critical	Main program
804	Changes to Preferences dialog	Improvement	critical	Main program
803	DentalEye crashes after annotations 3.2.1 BETA 6	Bug report	medium	Main program
801	Camera WIA upgrade to DE32 standard	Improvement	high	Plugin: Camera WIA
800	Medspace plugin upgrade for 3.2 req	Improvement	high	Plugin: Medspace
799	C-Takt plugin, 3.2 req check	Improvement	high	Plugin: C-takt
798	Export plugin, implement 3.2 requirements	Improvement	medium	Plugin: Export
797	Manual Import plugin, implement CAE & CAD support	Improvement	high	Plugin: Manual import
796	TWAIN plugin, implement CAE & CAD	Improvement	critical	Plugin: TWAIN
793	Export to DCZ/DEZ must not include other files than DICOM	Bug report	critical	Main program
791	No import for Excel, Word	Bug report	medium	Main program
790	Erratic behavior during DICOM import to new patient	Bug report	critical	Main program
789	DICOM import should show a progress bar	Improvement	high	Main program
788	Database connection failure not handled gracefully	Design error	critical	Main program
787	X-image after capture in session with no imagestore connection.	Bug report	critical	Main program
786	Importing CT files in Pro edition saved DCM file with JPG extention	Bug report	medium	Main program
781	DEApprove must not be active if there is no license	Improvement	medium	Tool
779	Remove "tabs" that show templates	Improvement	medium	Main program
778	IP address not accepted in image store folder setting	Bug report	medium	Main program
777	Excess items in Program Files	Bug report	medium	Installer
776	Excess item in start menu	Bug report	low	Installer
775	Patient ID label not hidden when scrolling with mouse wheel in CT	Bug report	medium	Main program
773	Error not logged in Preferences	Bug report	high	Main program
772	License generator - new version and new database for 3.2	Improvement	critical	License system
771	Image import browse dialog needs upgrade	Improvement	medium	Main program
766	Presentation view behavior	Design error	critical	Main program
762	MVRQ0066: Allow users with "configure and view audit trail" to access Audit trail options and log.	Bug report	medium	Main program
761	MVRQ0065: X-ray Control Book	Improvement	medium	Main program
757	MVRQ0061: Enable File Format drop-down box even if no checkboxes are selected on the Image Export dialog.	Bug report	medium	Main program
756	MVRQ0060: Display customizable alert when saving a series as unapproved	Improvement	medium	Unspecified
751	Include Command line link in 3.2 package	Improvement	high	PMS Link
750	Drag image to "NONE" frame must not delete existing anatomic info	Bug report	high	Main program
749	Edit Number	Improvement	medium	Unspecified
748	Patient Selector	Improvement	high	Tool

ID	Summary	Type	Priority	Component
747	SQL backup	Improvement	medium	Tool
745	Include DCM Store plugin in 3.2 package	Improvement	medium	Unspecified
744	Series of a certain type (IO, VL etc) must not allow new images of another.	Design error	critical	Main program
743	Remove checkboxes to hide original series and auto create IC copy	Design error	critical	Main program
740	Include Approval tool in 3.2.1	Improvement	critical	Tool
734	Help system in 3.2	Improvement	critical	Documentation
733	Cancel SaveAs on new IC with added image should not remove new IC from list!	Bug report	medium	Main program
724	TWAIN plugin stops working	Bug report	high	Plugin: TWAIN
718	Long patient names may cause series not to upload to DICOM server	Bug report	low	Main program
711	Release package identifier for 3.2	Improvement	critical	Unspecified
707	Red corner disappears when image is zoomed	Bug report	high	Main program
706	3.1 detection by installer	Improvement	critical	Installer
701	CAD & CAE support in e2v EV71JU21X plugin	Improvement	critical	Plugin: EV71JU21X
698	CAD & CAE support in Sirona plugin	Improvement	medium	Plugin: Sirona
666	DICOM storage requires DE database to provide redundancy	Improvement	critical	Documentation
665	Instructions on manual patient selection	Improvement	critical	Documentation
664	"Recycle bin" for deleted images	Improvement	critical	Main program
663	Add folder ID check on image open	Improvement	critical	Main program
662	Plugins in 3.2.1	Improvement	critical	Unspecified
659	Images deleted after scan using developer	Bug report	critical	Unspecified
654	Change manipulation indicator	Improvement	critical	Main program
653	Change ruler tools to mitigate risk	Improvement	critical	Main program
652	Approval workflow changes	Improvement	critical	Main program
651	Expired licenses	Improvement	high	License system
650	System information tool	Improvement	high	Tool
649	Display patient name and ID during capture	Improvement	critical	Main program
648	Error logging	Improvement	high	Main program
646	Revision of the License generator for 3.2	Improvement	critical	License system
645	License file import improvement	Improvement	medium	License system
643	AutoImport plugin, implement Enable/Disable support.	Improvement	high	Plugin: Autoimport
639	Monitor calibration image should be DPI adapted [DPI awareness]	Design error	high	Main program
583	Plugins registry reader function may be limited to 100 char strings	Design error	medium	Unspecified
560	C-Takt export of inverted and approved image does not keep inversion	Bug report	low	Main program
397	Use OS default font in GUI.	Improvement	low	Main program
367	Printer error 482 when printing IC to DICOM printer	Bug report	low	Main program
363	Problems with upgrade/install of some plugins	Bug report	insignificant	Installer
312	If Approval tool is closed in minimized state it will open minimized next time	Bug report	low	Tool
295	Stop supporting "Eternal" licenses.	Improvement	medium	Main program
48	Upgrade SUNI SDR plugin for DE3.2	Improvement	critical	Plugin: SUNI
46	Morita plugin upgrade for 3.2	Improvement	critical	Plugin: Morita
45	Kodak/Carestream plugin upgrade for DE3.2	Improvement	critical	Plugin: Kodak
44	Gendex plugin upgrade for DE3.2	Improvement	medium	Plugin: Gendex
42	Schick plugin for DE3.2	Improvement	medium	Plugin: Schick
41	Upgrade Planmeca plugin for DE3.2	Improvement	medium	Plugin: Planmeca
40	Dürr plugin	Improvement	critical	Plugin: Dürr
39	Camera WIA various changes	Improvement	medium	Plugin: Camera WIA
28	Cache and thumbnails	Design error	high	Main program
4	New export dialog	Improvement	critical	Main program